



# MULTI DRUG 6 PANEL TEST

AMP/THC/MOP/MET/BZO/COC (Urine)

## 6in1

**SIMPLE TO USE** | **HIGH ACCURACY** | **FAST READOUT** | **DETECT DRUG ABUSE UP TO 3-5 DAYS**

### Product Information:

The AyoTes Multi Drug 6 Panel Test (AMP/THC/MOP/MET/BZO/COC) (Urine) is a rapid chromatographic immunoassay for the qualitative and simultaneous detection of one to six of the following drugs in a variety of combinations in human urine. The designed cutoff concentrations and direct calibration for these drugs are as follows

Test	Calibrator	Cut-off (ng/mL)
AMP	d-Amphetamine	1000
BZO	Oxazepam	300
MET	Methamphetamine	500
MOP	Morphine	300
THC	11-nor- $\Delta^9$ -THC-9-COOH	50


Cat no.	Specimen	Shelf Life	Storage
IDAT-AS2023011	Human Urine	24 Months	2-30°C



### Important

For the adulteration tests, visually compare the color of the reaction pad with the color card, and the results should be read at 2 minutes. Do not interpret the results after 5 minutes.

### How to Use:

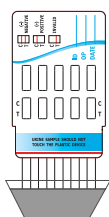


#### STEP 1

Dip the sample pad area of the dipstick strip or the dipstick card in the urine specimen submerging only up to the "MAX" mark of the dipstick strip or the edge of the dipstick card.



2-5 MINUTES

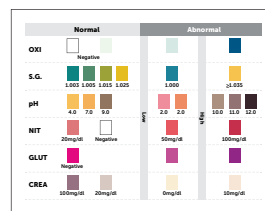

#### STEP 2

Read the results at 2-5 Minutes

### Results interpretation:




2 MINUTES



Positive



Negative

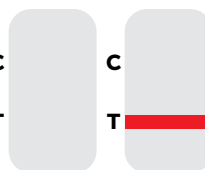

Invalid



**Important:**  
Do not interpret the results after 5 minutes

For orders & inquires

**PT Fokus Keluarga Sehat**

Jl. KH. Mansyur No. 129-130, RT.9/RW.3. Karet Tengsin, Tanah Abang District, Central Jakarta City  
Special Capital Region of Jakarta 10250, Indonesia

0813-8005-8788

info@fkssehat.com